


# An easy and quick way to share your blood glucose results with your healthcare professional

Become a CONTOUR® DIABETES app user and connect with your healthcare professional remotely\*

1

**Download  
CONTOUR®  
DIABETES app**

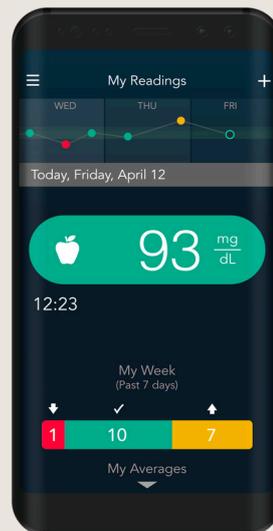

Free download



Download on the  
App Store

GET IT ON  
Google Play

For a list of compatible devices,  
please visit  
[compatibility.contourone.com](http://compatibility.contourone.com)

2

**Create  
an account**



3

**Pair your compatible  
CONTOUR® meter**



For more information visit:  
[sa.diabetes.ascensia.com](http://sa.diabetes.ascensia.com)

\*Patients need to be registered in GlucoContro.online and approve sharing their data with their healthcare professional

## Simple onboarding to enable remote blood glucose assessment with your healthcare professional

Wait for the invitation from your healthcare professional

Invitation via email

or

Invitation via QR code

Receive an email invitation and click on the link\*



1

Scan the QR code generated during the visit

**Add new Patient**

Manual      QR Code

**Mandatory**

First Name      Last Name

Enter first name      Enter last name

Country

United States of America

**Optional**

I want to send an e-mail invitation to this patient

E-Mail address

Enter E-mail address

**Add new Patient**

**Add new Patient**

Manual      QR Code

2

Log in with your CONTOUR®DIABETES app username and password

Patient      Professional

**Contour** Enabling with you To log in to Your account use Contour Cloud credentials.

E-mail address

Enter e-mail address

Password [Forgot your password?](#)

Enter password

Remember me

**Sign in**      Don't have an account? **Sign up**

3

Fill out your information



Now your blood glucose data is accessible for your healthcare professional via GlucoContro.online

\*Healthcare professional to initiate the invitation



**Distributor: Batterjee National Pharmaceutical**  
Phone: + 966 12 661 3091  
Call Center: +800 116 6110  
Fax: + 966 12 669 1625  
Email: [nassimuddin.cco@ascensia.com](mailto:nassimuddin.cco@ascensia.com)  
Call Center Hours: 9 a.m. - 5 p.m. / 5 days a week  
[sa.diabetes.ascensia.com](http://sa.diabetes.ascensia.com)